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Correct use

The mercury-free vet thermometer is an analogue maximum thermometer and is used to determine the body temperature of animals and humans in a temperature range from 35° to 42°C. Any other use is deemed as incorrect.

Safety instructions

Caution!

Danger of physical injury and material damage caused by incorrect use.

- Before use, read the operating instructions carefully.
- Only use for the correct intended purpose.
- Store the thermometer at an ambient temperature between 20 °C and 42 °C.
- Do not drop the thermometer and do not subject it to sudden shocks.
- Do not bend the tip of the thermometer or bite it.
- Keep out of the reach of children.
- Do not use thermometers with damaged glass casings.

Operation

1. Disinfect the thermometer.
2. Ensure that the ambient temperature is between 20 °C and 30 °C. If necessary, store the thermometer for 1-2 hours within this temperature range.
3. Ensure that the column of liquid is below the first numbered graduation mark (37 °C). If necessary, hold the thermometer at the top end and shake with 15 to 20 quick movements from the wrist.
4. Place the thermometer into the mouth (under the tongue) or max. 2 cm into the rectum.
5. After around 3 minutes, read off the body temperature using the column of liquid in the capillary tube.

Cleaning and disinfection

Use cold water or disinfectant to clean and disinfect the thermometer. Do not use solvents, petrol or benzene.

Disposal

The thermometer can be disposed of in domestic waste. If the thermometer breaks, mop up any leaked liquid using a fabric cloth moistened with soapy water.

Technical data

Art. No.	2125
Measuring range	36 – 43 °C
Measuring accuracy	+/- 0,1 °C

The thermometer complies with the requirements of Directives 93/42/EC and EN 12470 1:2000 part 1.

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